
BWISE

BWISE Lipotul

Lipo-Colloid Net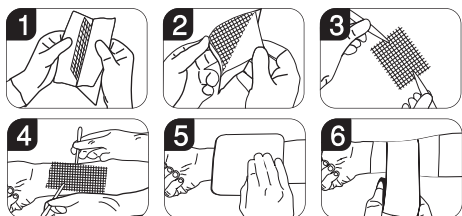

CH REP

CMC Medical Devices GmbH
Rigistrasse 3, 6300 Zug, Switzerland

CH

Giulia Medical SA
Roosstrasse 53, 8832 Wollerau (SZ), Switzerland

EC

BAP Medical B.V.
Laan van Westenenk 64, NL-7336 AZ Apeldoorn,
The Netherlands

UK

Griffiths & Nielsen Ltd.
Maydwell Avenue, Stane Street, Slinfold, Horsham
RH13 0GN, United Kingdom

E C REP

E C Rep Ltd.

5 Fitzwilliam Square East,
Dublin 2, D02 R744, Ireland.
Tel: +353 1 2 544 944
Email: info@ecrep.ie

Pharmaplast S.A.E.
Amria, free zone 23512,
Alexandria, Egypt.
www.pharmaplast-online.com

INSTRUCTION FOR USE

BWISE Lipotul

Description:
BWISE Lipotul is a Lipo-Colloid non-adherent wound contact layer. BWISE Lipotul is composed of polyester mesh impregnated with superabsorbent powder and petrolatum.

Intended Purpose:
BWISE Lipotul is indicated for management of non to low exuding wounds at the granulation and epithelization phase. It could be used for: burns, traumatic wounds, abrasions, post-operative wounds, leg ulcers, pressure ulcers, diabetic foot ulcers, and epidermolysis bullosa.

Precautions:
1- BWISE Lipotul can adhere to Latex surgical gloves, so it's preferred to moisten the gloves with saline or use sterile tweezers to facilitate handling of the dressing.
2-The product is non-occlusive, so it could be used over infected wounds under medical supervision, and the dressing must be changed daily.
3-In case of using BWISE Lipotul in deep or fistula wounds, a section of the dressing should be left visible to enable easy removal.
4-The product is sterile, don't use if the pouch is damaged.
5-The product is a single use device, re-use may lead to the risk of infection.
6-Don't use in a hyperbaric chamber.
7-In case of development of skin irritation, erythema or overgranulation, please consult your healthcare giver.

Contraindications:
Don't use in case of sensitivity to any of the product ingredients.

Instructions for Use:

- Cleanse the wound as per local procedures, then rinse with saline solution.
- Dry the surrounding skin carefully.
- BWISE Lipotul can be cut to fit the wound area and shape, if required.
- Choose a dressing which is slightly larger than the wound.
- Remove the dressing's protective liner.
- Apply the dressing in intimate contact with the wound base, ensuring the entire surface is covered.
- Cover with a secondary dressing suitable for the level of wound exudate.
- Then cover with an appropriate retention bandage. The retention bandage shouldn't be occlusive, so as not to hinder breathability of the dressing.
- Generally the dressing could left in place up to 7 days, depending on exudates level and clinical condition of the wound.
- For Epidermolysis bullosa patients, the dressing should be changed every 1 to 3 days.
- To remove BWISE Lipotul, use tweezers, or forceps or a gloved hand to gently lift the dressing away.

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Pharmaplast		Amria 23512, Alexandria - Egypt www.pharmaplast-online.com		Leaflet artwork template no: I-TM /AR/001-5 (Version 5)			
Customer : B-wise		Product name : BWISE Lipotul					
Issue date : 20-04-2024		Prepared by : Nader Nabil Zaki					
Colors :		Dimension : xx x xx					
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by	
						Artwork manager	RA Department
1	N/A	New Issue	N/A				