
BWISE Silicone Interface Layer (1 - Sided)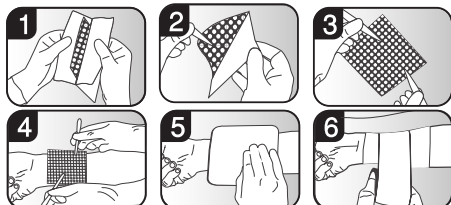

CH REP
CMC Medical Devices GmbH
Rigistrasse 3, 6300 Zug, Switzerland

EC
BAP Medical B.V.
Laan van Westenenk 64, NL-7336 AZ Apeldoorn, The Netherlands

UK
Griffiths & Nielsen Ltd.
Maydwell Avenue, Stane Street, Slinfold, Horsham RH13 0GN, United Kingdom

EC Rep Ltd
5 Fitzwilliam Square East,
Dublin 2, D02 R744, Ireland.
Tel: +353 1 2 544 944
Email: info@ecrep.ie

Pharmaplast S.A.E.
Amria, free zone 23512,
Alexandria, Egypt.
www.pharmaplast-online.com

INSTRUCTION FOR USE BWISE Silicone Interface Layer

Description:
BWISE Silicone Interface Layer is a porous, atraumatic wound contact layer. The silicone coating is slightly tacky, which facilitates the application and retention of the dressing to the skin surrounding the wound. This gentle adhesion also tends to prevent maceration by inhibiting the lateral movement of exudate from the wound on to the surrounding skin. Soft Silicone ensures removal of the dressing without pain or trauma to the delicate new tissues. BWISE Silicone Interface Layer is not absorbent, but contains apertures or pores that allow the passage of exudate into a secondary absorbent dressing. BWISE Silicone Interface Layer is single side coated with Silicone adhesive.

Intended Purpose:
BWISE Silicone Interface Layer are used for management of wounds where adherence to the underlying tissues should be avoided. Typical applications include skin tears or abrasions, surgical excisions, first and second degree burns, blistering conditions, lacerations, partial and full thickness grafts, leg and pressure ulcers, diabetic ulcers, venous and arterial ulcers and skin damage following radiotherapy or steroid therapy.

- Precautions:**
- Wounds should be inspected for signs of infection according to clinical practice guidelines. Consult a healthcare professional for the appropriate medical treatment.
 - When BWISE Silicone Interface Layer is used on burns treated with meshed grafts, avoid placing unnecessary pressure upon the dressing.
 - When BWISE Silicone Interface Layer is used after facial resurfacing, avoid placing pressure upon the dressing, lift and reposition the dressing at least every second day.
 - When BWISE Silicone Interface Layer is used for the fixation of skin grafts and protection of blisters it is recommended that, the dressings may be partially overlapped, ensuring that the pores are not blocked.
 - Single use device. Reuse may cause cross contamination.

- Instructions for Use:**
1. Clean the wound and dry the surrounding skin thoroughly.
 2. Remove the release liners from both sides of the dressing.
 3. A dressing should be selected that overlaps the wound margin by at least two centimeters and, if necessary, this may be cut to size or shape before removal of the protective films. If more than one piece of BWISE Silicone Interface Layer are required, the dressings may be partially overlapped, ensuring that the pores are not blocked.
 4. Smooth into place and ensure good sealing with the surround wound.
 5. Cover with absorbent secondary dressing according to the intensity of exudates e.g. BWISE Foam or BWISE Superabsorbent in case of heavy exudates.

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