BWISE

EN

INSTRUCTION FOR USE
BWISE Super Absorbent Dressing

Description
BWISE Super Absorbent Dressing Non-adherent is an absorbent wound dressing that helps to maintain a moist environment at the wound surface. It comprises an absorbent core for absorb and retain exudates, the absorbent core is enclosed in hydrophobic breathable layer and liquid permeable layer. Super Absorbent Dressing absorbs and retains exudates and MMPs into the absorbent core, to reduce the risk for maceration, and encourage wound healing. Dressing material contents: Polypropylene, Polyethylene, Cellulose, Polyacrylate.

Indications
BWISE Super absorbent dressing is intended for used on moderate to heavily exuding wounds, such as: leg ulcers, pressure ulcers, diabetic foot ulcers and surgical wounds.

Instructions for use
Super absorbent dressing can be used by health care professionals and lay persons under supervision of health care professionals.

1. After wound is cleaned and sterilized, the appropriate size dressing is selected to allow for sufficient space to the wound edges.

2. BWISE Super absorbent dressing can be used both as a primary dressing or as a secondary dressing. When used as a primary dressing, BWISE Super absorbent dressing must be in close contact with the wound bed to ensure wound fluid is absorbed. Secure accordingly with bandage, tapes or compression.

3. The frequency of changing dressing depends on the amount of exudates.

4. Dressing removal can be facilitated by moistening the dressing with saline solution.

Precautions and warning

- The product is single use. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance, and Cross-contamination may occur.
- Sterile. Do not use if sterile packaging is damaged or opened prior to use. Do not re-sterilize.
- Do not cut the dressing, the cover must remain intact.
- In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and following a clinician's recommendation.

Contraindications

- Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressing swells during absorption.
- Do not use on third-degree burns.
- BWISE Super absorbent dressing should not be used on persons with known allergy or hypersensitivity to any of its components.

Storage

Keep in a clean and dry place in specified temperature range and away from sunlight;

Disposal
Dispose should be handled according to local environmental procedures.

Other information
If any serious incident has occurred in relation to the use of BWISE Superabsorbent Dressing, it should be reported to the manufacture and competent authority of the Member States.

Shelf life: 3 years from date of manufacture

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