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## BWISE INSTRUCTION FOR USE

### BWISE Super Absorbent Dressing (Type I)

**Product description**  
Super Absorbent Dressing is indicated for the acute and chronic wounds with moderate to heavily exudate, such as diabetic foot ulcers, pressure ulcers, leg venous and arterial ulcers, first and second degree burns, post-operative wounds and traumatic wounds, absorbs large amount exudates, provides a moist environment for wound bed and facilitates healing process. This is a professional use and single use device. The dressing can be left in place for up to 7 days, depending on the degree of exudation. However, it should not be used continuously for more than 30 days (in order to be replaced by the same device immediately, the device can be stopped from use, which will be regarded as an extension of the continuous use of the device. 30 days are determined according to this principle).

**Intended purpose**  
Super Absorbent Dressing (Type I: Thick type and five layers structure) is a highly absorbent wound dressing for the management of moderate to highly exuding wounds. It was designed to absorb wound exudate and provide a moist wound healing environment to promote wound healing.

**Intended user**  
The product shall be used by or under the supervision of a healthcare professional.

**Indications**  
Super Absorbent Dressing (Type I: Thick type and five layers structure) is indicated for the acute and chronic wounds with moderate to high levels or exudate, such as, diabetic foot ulcers, pressure ulcers, leg venous and arterial ulcers, first and second degree burns, postoperative wounds, and traumatic wounds.

**Contraindications**  
1. The Super Absorbent Dressing cannot be used on dry wounds, heavy bleeding wounds, third degree burns and surgical implantation;  
2. Check the wound for signs of infection before use, if infection occurs, see a health care professional;  
3. Do not use on the patient with a known hypersensitivity to the product itself or to its components;  
4. Do not use in tunnelling wound as the product can expand considerably when wound exudate is absorbed.

**Adverse reaction**  
The human bodies are well tolerated to Super Absorbent Dressing products, however, after combined the available clinical data and postmarketing data of competitive device and benchmark devices, the following situations are identified, and they are very rare:  
-Mechanical skin damage and resulting pain.  
Pain from mechanical skin damage or epidermal dryness may occur; other allergic reactions are less likely.

**Instructions for Use**  
First step: Select a size of dressing enough to overlap the wound edges by at least 2-3cm.  
Second step: Gently apply the dressing directly on the wound site with the white hydrophilic side of dressing onto the wound surface.  
Third step: The dressing shall be fixed by an appropriate secondary dressing, such as Adhesive tape, elastic bandage or pressure bandage.

**Note:**  
The dressing can be left in place for up to 7days, but wearing time should be depended on level of exudate, because of excellent fluid handling capability of the dressing, it may produce pressure and require change.  
**Warning and Precautions**  
1) Do not reuse. Reusing will cause cross-contamination.  
2) Do not use if package is damaged or open.  
3) For non-sterilized products, sterilize before use.  
4) Check the wound for signs of infection before use, if infection occurs, see a health care professional.

**Storage and transport conditions**  
The product should be stored in warehouse under normal room condition, temperature and humidity of room will not be specified strictly but require cool and dry. Direct sunlight shall be avoided. The transportation should prevent the weight, sun and rain and rain dripping.  
**Disposal**  
Disposal should be handled according to local environment procedures.

**Other information**  
1.Patients can undergo an MRI scan while using this dressing.  
2.If have any serious incident that has occurred in relation to the device, please reported to the manufacturer and the competent authority.

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